ES

IT

DE

EN

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**Indications for Use:**  
For sealing pits and fissures on teeth showing no decayed lesions.

**Contraindications:**  
Alpha-dent® Light Cure Pit & Fissure Sealant is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

**Warning:**

• Avoid contact of etchant with soft tissue, eyes and skin. Alpha-dent® etchant and Alpha-Etch37® contain phosphoric acid which is corrosive. In case of contact with soft tissue, eyes or skin immediately flush with water for at least 15 minutes. Seek immediate medical attention.

**General Precautions:**  
• U.S. Federal Law restricts this device to sale by or on the order of a dentist.

• Review instructions and SDS ([www.dentaltech.com/resource](http://www.dentaltech.com/resource)) for all provided materials prior to beginning treatment. Use only as directed.  
• Do not use in the presence of eugenol based materials as complete setting may not occur.  
• Applicator tips and brushes (if supplied) are disposable and intended for single patient use.  
• Always replace cap after each use.

**Adverse Reaction:**  
Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

**Instructions for Use:**

1. Prepare the surface to be treated. The recommended method for preparing the occlusal surface is with an air slurry polisher with sodium bicarbonate slurry. If performing a prophylaxis on the occlusal surface, prophy paste which contains oil or fluoride should not be used as it may leave a residue which can inhibit bonding.  
2. Rinse thoroughly with water to remove all residual preparation material from the pits and fissures.  
3. Isolate the teeth to be sealed with cotton rolls and/or a rubber dam.

4. Dry thoroughly with moisture-free and oil-free compressed air.  
5. Apply Alpha-dent® etchant or Alpha-Etch37® to the occlusal surface following the etchant instructions. Wait for 15 seconds; never leave etchant on tooth for longer than 60 seconds. Lightly agitating the etchant during its application can enhance the action.

6. Rinse thoroughly with water. Wet cotton should be replaced. Salivary contamination must be avoided at this point. If contamination occurs repeat steps 4, 5, and 6.

7. Dry the etched surfaces thoroughly with moisture-free and oil-free compressed air. The dry etched surface should appear dull and frosty white. If surface does not repeat steps 4, 5, and 6.

8. Apply a thin, uniform coat of Alpha-dent® Light Cure Pit & Fissure Sealant using the brushes (if supplied) or other suitable instrument to the pits and fissures.

9. Using a light-curing unit (minimum output of 600mW/cm<sup>2</sup>), cure all coated surfaces for at least 20 seconds. Thicker or larger applications may require additional curing time.

10. Inspect the selected surfaces. If coverage is incomplete, apply additional sealant. If contamination of the etched surface has occurred, repeat steps beginning with step 4 above.

11. Check occlusion and adjust as required.

**Storage:**

• Refrigeration is recommended. Store material between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.  
• Do not expose to elevated temperatures or intense light.  
• Keep containers closed when not in use.

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**alpha-dent®**  
**LIGHT CURE**  
**PIT & FISSURE SEALANT**  
LIGHT CURE

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