ES

Descripción:

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EN

Description:

Alpha-dent[®] etchant liquid and **Alpha-Etch 37**[®] gel contain 37% phosphoric acid for use on both dentin and enamel. The liquid viscosity of **Alpha-dent**[®] etchant achieves greater surface contact. The gel viscosity of **Alpha-Etch 37**[®] provides ideal control to prevent the material from flowing onto exposed surrounding surfaces. Both etchants rinse away clean without leaving any residue, and are available in a blue or green color that makes it easily distinguishable from the tooth structure.

Indications for Use:

Acid etching of enamel and dentin.

Contraindications:

Alpha-dent[®] and **Alpha-Etch 37**[®] etchants are contraindicated for use on soft tissue including oral mucosa, skin, eyes and dental pulp tissue.

Warning:

- Avoid contact of **Alpha-dent**[®] and **Alpha-Etch 37**[®] etchant with soft tissue, eyes and skin. Both contain phosphoric acid which is corrosive. In case of contact with soft tissue, eyes or skin immediately flush with water for at least 15 minutes. Seek immediate medical attention.

General Precautions:

- U. S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.
- Proper personal protection equipment must be used to prevent etchant from coming in contact with soft tissue, eyes or skin.
- Alpha-dent**[®] etchant is intended to be applied to the surface to be etched with a brush or other suitable instrument.
- Alpha-Etch 37**[®] is supplied within a syringe and is intended to be dispensed with the supplied single use applicator tips.
- Always replace the cap after each use.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:**Gel Syringe Preparation**

- Remove the cap from the syringe and save. Attach a new provided applicator tip.
- Prior to application, dispense a small amount of **Alpha-Etch 37**[®] onto a dispensing pad or other suitable surface to verify the flow, and that the applicator tip is properly attached prior to intraoral use.

Liquid Bottle Preparation

- Remove the cap and dispense **Alpha-dent**[®] etchant onto a dispensing pad or other suitable surface.
- Load the brush or other suitable instrument with **Alpha-dent**[®] etchant for application. Be sure to maintain a controllable amount of etchant liquid on the brush or other suitable instrument to avoid unwanted contact of etchant with soft tissue, eyes and skin.

Acid Etching Procedure

- Prepare the surface to be etched in the usual manner.
- Isolate and dry tooth thoroughly with oil-free compressed air.
- Apply etchant to the prepared enamel for 15 seconds, or if applied to dentin etch for 10 seconds. Lightly agitating the etchant during its application can enhance its action. Never leave the etchant on the tooth for more than 60 seconds.
- Rinse thoroughly with water. The dry etched surface should appear dull and frosty white. If the surface to be etched does not, then repeat from step 3.

Storage:

- Store material between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer packaging for expiration date.
- Keep container closed when not in use.

DENTAL
TECHNOLOGIES

ALPHA-DENT[®] / **ALPHA-ETCH 37**[®]
ETCHANT
37% PHOSPHORIC ACID

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