ES

Descripción:

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EN

Description:

Alpha-dent® Self Cure Bonding Adhesive is a versatile adhesive consisting of a base and catalyst. **Alpha-dent®** Self Cure Bonding Adhesive is fast, easy to apply and offers excellent bond strengths to dentin and enamel.

Indications for Use:

Alpha-dent® Self Cure Bonding Adhesive is a dental adhesive designed to bond self cured composite restorative materials to enamel and dentin.

Contraindications:

Alpha-dent® Self Cure Bonding Adhesive is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

Warning:

• Avoid contact of etchant with soft tissue, eyes and skin. **Alpha-dent®** etchant and **Alpha-Etch37®** contain phosphoric acid which is corrosive. In case of contact with soft tissue, eyes or skin immediately flush with water for at least 15 minutes. Seek immediate medical attention.

General Precautions:

• U. S. Federal Law restricts this device to sale by or on the order of a dentist.
• Review instructions and SDS (www.dentaltech.com/resource) for all provided materials prior to beginning treatment. Use only as directed.
• Be certain to avoid cross contamination of the base and catalyst. The presence of either product in the other will result in hardening of the adhesive in the bottles.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:

1. Prepare the tooth. Isolate and dry the tooth thoroughly.
2. If using **Alpha-dent®** etchant or **Alpha-Etch37®**, apply etchant to the prepared enamel for 15 seconds, or if applied to dentin etch for 10 seconds. Lightly agitating the etchant during its application can enhance its action. Never leave the etchant on the tooth for more than 60 seconds.
3. Rinse teeth thoroughly with water to remove all traces of the etchant. Re-isolate and dry with oil-free and moisture-free compressed air. The surface should appear dull and frosty white. If not, repeat step 2. It is important that the tooth be completely dry before proceeding.
4. Dispense equal amounts, one or two drops of each **Alpha-dent®** Bonding Adhesive catalyst and base, onto a mixing pad of other suitable surface. Mix thoroughly for 5 to 10 seconds.
5. Thoroughly saturate a brush tip and generously apply one uniform layer onto the prepared tooth. **Alpha-dent®** Bonding Adhesive will begin to harden within 30 seconds from the start of mixing.
6. After 45 seconds, placement of the self cure restorative material can proceed over the **Alpha-dent®** Bonding Adhesive without waiting for the bonding adhesive to set completely. Complete setting of **Alpha-dent®** Bonding Adhesive will occur 90 seconds from the start of mixing. Refer to manufacturer's instructions for placement, cure and finishing of the restorative materials.

Storage:

• Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.
• Keep containers closed when not in use.

Composition:

• Methacrylates, Initiators, Inhibitor.

Disposal:

• Dispose of contents/container according to local and national regulatory requirements.

Vigilance:

• If a serious incident occurs with this medical device, you must report it to the manufacturer and to the competent authority in which the user and/or patient is located.

Summary of Safety and Clinical Performance:

• Can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>).

Validity:

• Upon publication of this instruction for use, all previous versions are superseded.

DENTAL
TECHNOLOGIES

alpha-dent®
SELF CURE BONDING
ADHESIVE

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