ES

Descripción:

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EN

Description:

Alpha-Dent® Light Cure Bonding Adhesive is designed for bonding composite resin materials to dentin and enamel. The ideal viscosity allows the material to penetrate into microtubules to enhance mechanical adhesion.

Indications for Use:

Alpha-Dent® Light Cure Bonding Adhesive is a dental adhesive designed to bond light cured composite restorative materials to dentin.

Contraindications:

Alpha-Dent® Light Cure Bonding Adhesive is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

Warning:

- Avoid contact of etchant with soft tissue, eyes and skin. **Alpha-dent**® Etchant and **Alpha-Etch37**® contain phosphoric acid, which is corrosive. In case of contact with soft tissue, eyes, or skin immediately flush with water for at least 15 minutes. Seek immediate medical attention.

General Precautions:

- U.S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.

Adverse Reactions:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:

- Prepare the tooth. Isolate and dry the tooth thoroughly.
- If using **Alpha-Dent**® Etchant or **Alpha-Etch37**® etch enamel for 15 seconds or if applied to dentin etch for 10 seconds; never leave etchant on tooth for longer than 60 seconds. Lightly agitating the etchant during its application can enhance its action.
- Rinse thoroughly with water and the tooth must be dried completely. The dry etched surface should appear dull and frosty white. If the surface to be etched does not, then repeat step 2.
- Dispense two drops of **Alpha-Dent**® Light Cure Bonding Adhesive onto a dispensing pad or other suitable surface. Thoroughly saturate a brush tip and generously apply one uniform layer onto the prepared tooth.
- Using a visible light-curing unit (minimum output of 600 mW/cm²) cure the bonding adhesive for 20 seconds. The surface should be glossy. If not, repeat steps 4 through 5.
- Place the light cure restorative material over the cured **Alpha-Dent**® Light Cure Bonding Adhesive. Refer to manufacturer's instructions for placement, cure, and finish of the restorative material.

Storage:

- Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.
- Keep container closed when not in use.

Composition:

- Methacrylates, Initiator, Inhibitor.

Disposal:

- Dispose of contents/container according to local and national regulatory requirements.

Vigilance:

- If a serious incident occurs with this medical device, you must report it to the manufacturer and to the competent authority in which the user and/or patient is located.

Summary of Safety and Clinical Performance:

- Can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>).

Validity:

- Upon publication of this instruction for use, all previous versions are superseded.

DENTAL
TECHNOLOGIES

alpha-dent®
**LIGHT CURE
BONDING ADHESIVE**
LIGHT CURE

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