ES

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- Methacrylate, Initiatoren, Inhibitor.

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EN

Description:
Alpha-Bond® SE is a light cure, self-etch adhesive formulated to bond to multiple substrates for both direct and indirect bonding procedures. It's a universal adhesive that has the bond strength and durability of a total-etch adhesive while eliminating the etching step.

Indications for Use:

- Direct Bonding of Composite Resins and Compomers
- Veneers
- Composite, Ceramic and Metal repairs
- Indirect restorations: inlays, onlays, crown and bridge retainers
- Endodontic post cementation
- Direct Bonding of Composites and Core Build-up Materials

Contraindications:

Alpha-Bond® SE is contraindicated in patients with a history of allergic reaction to methacrylate resins.

Warnings:

- Alpha-Bond® SE** is flammable, keep away from sources of ignition.
- Avoid contact with eyes and soft tissue. In case of contact with eyes, immediately flush with copious amounts of water for at least 15 minutes while holding eyelids open. Seek medical attention. In case of skin contact, remove contaminated clothing and flush exposed area with water and follow by washing with soap if available.

General Precautions:

- U. S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.
- Always replace cap after each use.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:

Prepare the tooth. Isolate, wash and dry tooth with oil-free compressed air.

Direct Bonding Composite Resins and Compomers to Dentin/Enamel

- Shake **Alpha-Bond® SE** bottle.
- Dispense 2-3 drops of **Alpha-Bond® SE** into a well, or other suitable surface. Replace cap immediately after dispensing.
- Use a disposable applicator brush (not supplied) and apply a coat to the enamel/dentin surface with scrubbing motion for 10-15 seconds.
- Apply a second application with scrubbing motion 10-15 seconds. Avoid pooling of **Alpha-Bond® SE** on the enamel/dentin surface. Excess **Alpha-Bond® SE** can be removed with absorbent paper points.
- Gently apply air for at least 5 seconds to evaporate the solvent, making sure you do not apply too much air to thin the adhesive. When properly dried the adhesive layer will not move with air and the surface should appear shiny. If the surface is not shiny it may not be properly sealed and additional applications are recommended.
- Using a light-curing unit (minimum output of 600mW/cm2), cure **Alpha-Bond® SE** for 10 seconds.
- Place restorative material immediately and cure according to the instructions.

Indirect Restorations – Veneers, Crowns, Bridges, Inlays and Metal-based Restorations

- Follow steps 1-7 from the instructions for **Alpha-Bond® SE** direct bonding.

Veneers – Prepare the tooth and veneer according to the manufacturer's instructions. Then apply the luting cement to the veneer, seat the veneer, and cure the luting cement following the manufacturer's instructions.

Crowns, Bridges, Inlays, Onlays and Metal-based Restorations – Apply a luting cement to the restoration or prep, seat the restoration, and cure the luting cement according to the manufacturer's instructions.

Endodontic Post Bonding – Apply a luting onto the post, seat the post, and move the post slightly to avoid the possibility of trapped air. Then cure the resin cement according to the manufacturer's instructions.

Core Build Up Bonding – Apply **Alpha-Core® DC** according to the instructions.

Storage:

- Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.
- Keep container closed when not in use.

Composition:

- Methacrylates, Acetone, Water, Methacrylate phosphoric acid ester, Initiator, Inhibitor.

Disposal:

- Dispose of contents/container according to local and national regulatory requirements.

Vigilance:

- If a serious incident occurs with this medical device, you must report it to the manufacturer and to the competent authority in which the user and/or patient is located.

Summary of Safety and Clinical Performance:

- Can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>).

Validity:

- Upon publication of this instruction for use, all previous versions are superseded.

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